



QUICK USER GUIDE

AIRPORT TERMINAL &
AISLE WHEELCHAIRS



**Manufactured by Mobility
Plus Wheelchairs Ltd**

SWL 120kg
Australian Made





THANK YOU!



Thank you for your purchase of a wheelchair/wheelchairs from Mobility Plus and for placing your trust in us. In doing so, you are supporting Australian Manufacturing and an Australian Made product.

We ask that you read the following guide carefully and thoroughly upon receipt of your wheelchair/wheelchairs and to please observe all instructions and warnings.

Please note, the guidelines and representations within this guide may differ from your product. Any images or descriptions are intended to show you the designation of the most common and important components and features of our Airport Terminal and Aisle Wheelchairs. Mobility Plus reserve the right to make technical modifications. For further information, please contact your Mobility Plus dealer.



PRODUCT DESCRIPTION**Parts Overview; Terminal Wheelchair****1. Push Handles****2. Attendant Brake Lever****3. Backrest / Backrest Upholstery****4. Arm Rest****5. Chest Strap****6. Brake / Toggle Brake****7. Seat / Seat Upholstery****8. Rear Wheel****9. Calf Strap****10. Front Castor****11. Footplate**

Parts Overview; Aisle Wheelchair



1. Push Handles

2. Backrest / Backrest Upholstery

3. Attendant Brake Lever

4. Chest Strap

5. Armrest

6. Seat / Seat Upholstery

7. Brake / Foot brake

8. Calf Strap

9. Rear Wheel

10. Footplate

11. Front Castor

**PRE-USE SAFETY CHECK LIST**

- Check upholstery & any lap or chest belts are free from tears or damage
- Check that tyres are not worn and are correctly inflated
- Check that toggle brakes are functioning – when pushed into the forward (locked) position they hold firm against the wheels and prevent them from turning
- Check attendant lever brakes are functioning – when squeezed, they prevent the wheels from turning
- Check that any anti-tip wheels are in place
- Check that calf strap is in place
- Check that both armrests are in place (aisle chairs – facing in forward position) and free from damage

SAFE USE; QUICK GUIDE

- Before getting into the wheelchair or out of the wheelchair, apply both brakes & fold footplates up
 - Do not stand on footplates when getting in or out of the wheelchair
- *Aisle chairs only - To facilitate a side transfer, arm rests can be positioned to the rear of the wheelchair. Return to front before use.
- Ensure all lap belts or chest belts are safely secured
- Once seated in the wheelchair, ensure foot plates are folded down and feet are positioned on footplates
- Check that all personal items and items of clothing are free from the wheels (both front & rear)
- Do not allow more than one user at a time
- Brakes should be applied when the wheelchair is not being pushed or used
- Do not go too fast, turn too quickly or use on uneven surfaces
- Do not hang any bags or heavy items from the rear of the wheelchair
- Any attendant must keep both hands on the push handles to control the wheelchair
- Do not attempt to navigate stairs
- Take extreme caution when navigating any slopes or lips

WARRANTY:

Mobility Plus wheelchair frames carry a limited life-time warranty (6 years). Any alterations to the wheelchair's original script must be carried out by Mobility Plus or an authorised agent for warranty to apply against workmanship and materials.

All other parts carry a 12-month warranty against workmanship and materials.

Parts not covered by warranty include: upholstery, tyres, tubes and spokes.

Normal wear and tear is not covered by warranty.





GENERAL WARNINGS

Safety & Maintenance

It is important to keep your wheelchair/wheelchairs in proper working condition. Always inspect and maintain your Mobility Plus wheelchair. If you notice or suspect a problem, please have the wheelchair serviced or repaired before continuing to use the wheelchair. We recommend having wheelchair fleets serviced by an authorised Mobility Plus technician at least once every 3 months. If you fail to inspect and maintain your wheelchair/wheelchairs in a safe, working condition, there is a risk for sustaining a serious injury as a result.

Mobility Plus Terminal and Aisle wheelchairs are not intended to be used as a seat in a motor vehicle.

Making Changes to your Wheelchair/s

No tools are required for assembly. Your wheelchair is assembled by your authorised Mobility Plus service provider. Please use extreme caution when making any adjustments or changes to a Mobility Plus wheelchair. We recommend that only authorised Mobility Service technicians make any adjustments, changes or modifications to our wheelchairs. Modifications that have been completed by an unauthorised technician may void the warranty or cause serious injury.

